




## A Key Doctor's Diary: The Central Pillar for Facial Harmony

PART 2

**Aquamid® provides superior, lasting results in remodelling the nose using deep sub dermal filler.**

This significant and relatively new category of non-surgical cosmetic procedure is of particular interest in the Asian patient population seeking an Augmentation Rhinoplasty. Asian patients often desire contouring to create a bridge and reduce a flattened appearance.

Applications include augmentation of nasal bridge, lengthening of the dorsal spine of the nose, tip projection, straightening out minor nasal deviations, or correcting minor irregularities of projection.

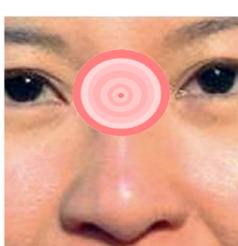

- Rhinoplasty is a cosmetic procedure used to enhance the size, shape or overall appearance of the nose.
- The nose is the centre of the face.
- When looking at someone's face, focus is on the nose first, before looking at the surrounding areas, following the pattern of circles from smaller to bigger.



**Augmentation Rhinoplasty gives more definition and contour and also gives the impression of a longer and more refined face overall.**




**Demonstrates a good patient outcome — before and after treatment to improve nasal projection and profile.** The nose can be a difficult area to inject. We must understand the local anatomy and be aware of the depth of injection. Generally only small volumes of product are required if well placed and if one learns how to contain the product.




**These images show the typical anatomy of the nose. There is the foundation of the nasal bones, the cartilaginous septum. Overlying this are muscles, nasalis, procerus chief amongst them.**

Note also the common vascular anatomy and take heed of the central corridor which is largely free of vessels of any substantial size. The nasal tip again is largely free of significant vessels. This then is the best option for placement of your needle or canula to avoid vascular compromise. Insertion via the tip where the alar cartilages meet centrally allows a straight advancement of the needle/canula.

### Nose [Skeleton] Anterolateral View

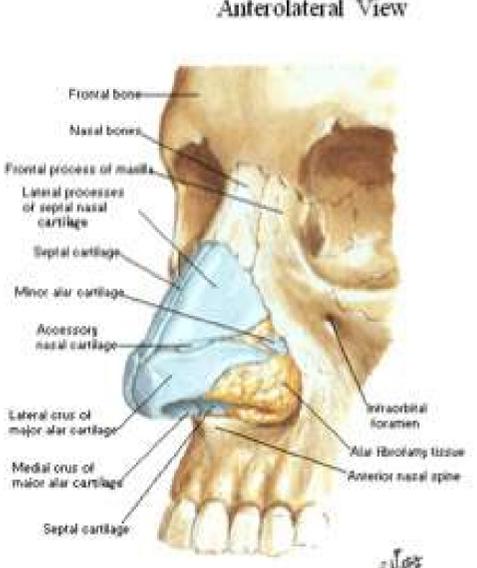

**Shows where I am demonstrating the loose skin on the centre of the nose.** As you can see in the video the skin of the dorsum of the nose is reasonably loose and allows us to form a low pressure corridor into which we can advance our needle/canula. This is important.

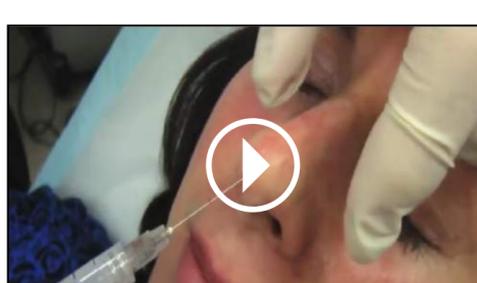

**Shows how I am advancing the needle in the anaesthetised patient.** This video shows a patient having combined facial procedures under general anesthetic, one of which is augmentation Rhinoplasty with Aquamid. Normally if only the Rhinoplasty is being performed local anaesthetic is all that is required.



**Demonstrates how non surgical Rhinoplasty can straighten out a nasal hump by filling the hollows either side.**

### Product is Important

Ideal product is minimally bio active, soft enough to be pliable on insertion, cohesive enough to hold its shape and be able to be integrated into host tissues.

**What do I use?** Aquamid.

**AQUAMID®**